

Rx Perioflush[®] LIQUID

FOR THE IRRIGATION OF PERIODONTAL POCKETS

INDICATIONS:

For use in rinsing and disinfecting periodontal pockets.

PRESENTATION:

RxPerioflush[®] is presented as a single pack containing 2ml (2g) of solution together with syringe applicators.

METHOD OF USE:

RxPerioflush[®] can be used according to individual need, either in acute or chronic conditions. Some 85% of users find considerable improvement after a single application to the infected pocket. The syringe and applicator have to be introduced with care so as not to further damage the gingiva. The

RxPerioflush[®] solution is injected with an up and down movement to ensure dispersal. Thereafter the patient should rinse their mouth with water.

RxPerioflush[®] can be warmed prior to application to minimise tooth sensitivity.

CONTRA INDICATIONS:

Known sensitivity to any constituents.

INTERACTIONS:

None known.

SIDE EFFECTS:

None if the preparation is used according to the instructions.

RxPerioflush[®] can be used in patients with ulcers, diabetes and pregnancy.

ADDITIONAL INFORMATION:

For use by dental surgeons.

Store at room temperature.

Keep out of the reach of children.

FORMULATION:

1. Hydrogen Peroxide
2. Chlorhexidine Gluconate
3. Phosphoric acid
4. Aroma

MANUFACTURER

Manufactured in the European Union

Dental Life Sciences (Mfg) Ltd.

Pierre Fauchard House Unit 3A

Wigan Enterprise Court, Manchester Road

Ince in Makerfield, Wigan WN2 2AG

United Kingdom

ISO 9001:2000

ISO 13485:2003

Package Insert English edition 1, 31st May 2006



DENTAL LIFE SCIENCES